## D - ANWEISUNG

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## GB - INSTRUCTIONS

Put on safety glasses, wear protective gown and gloves.

Disinfect the work space. In case of re-usage of the dispenser bucket, including the lid, these parts (REF 16074) must be cleaned and thermally (90°C) or chemically disinfected and dried afterwards.

Remove the lid of the dispenser and put the unpacked Petsorb® roll into the dispenser. Pull out a small piece of the innermost wipe from the middle (**picture 1**).

Slowly pour the solution in accordance with the manufacturer's instructions (usually 1,5 - 2 liter) spirally from the middle outwards over the roll. This will ensure a complete wetting (**picture 2**).

Pull the innermost wipe of the roll through the tear off slot of the dispenser lid and close the cover and the lid (**picture 3**).

For complete wetting of wipes, allow for soaking of minimum 30-60 minutes. As soon as the wipes are fully wetted, these can be used.

For max. distribution of solution, the wipe can be folded one or multiple times. Write down the type of solution, applied concentration and preparation date on the included batch label, and stick it on the dispenser (**picture 4**).

The concentration and the filling quality in accordance with the recommendation of the manufacturer of the cleaning or disinfection agent.

For the Petsorb® nonwoven rolls a proof of principle test had been conducted in accordance with 4-field-test VAH / EN 16615.

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Petsorb® is a registered trade mark of Innopart BV